



SAFE life™

# Hair Processor & Multi-Panel Rapid Hair Test

Where Every Strand Holds the Key to a Safer Tomorrow!

When it is important to identify long-term drug use quickly, there is no better choice than the SAFE life™ Hair Processor, designed for on-site testing, with a comprehensive menu and an impressive **above 95% accuracy rate when compared with the gold standard LC-MS/MS testing**. No more waiting for negative results – send only positive samples for confirmation, saving time and cost.

## Features & Benefits

- Suitable for employment and insurance testing
- Results in 5 minutes
- Choose from a comprehensive menu of accurate and reliable drug tests
- Customizable panel options
- Private and observed sample collection
- Minimizes the risk of adulteration, ensuring result reliability
- Captures long-term drug use history from 2 weeks to 90 days



## Available drug test menu with customizable panel options.


- |                         |  |                       |
|-------------------------|--|-----------------------|
| • Amphetamines (AMP)    | • Fentanyl (FTY)                       | • Opiates (OPI)       |
| • Benzoylcegonine (BE)  | • Methamphetamine (MET/mAMP200)        | • Oxycodone (OXY)     |
| • Benzodiazepines (BZO) | • Methamphetamine (MET/mAMP500)        | • Phencyclidine (PCP) |
| • Cocaine (COC)         | • Methylenedioxymethamphetamine (MDMA) |                       |

## Procedure


Efficient, Accurate, On-site simplicity for drug testing confidence.




Collect the hair sample, place it into the processing tube, and screw the cap.

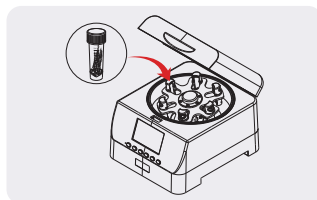

Insert the processing tube into the SAFE life™ Hair Processor and press the start button.

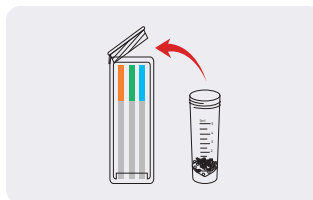

When processing is complete, remove the processing tube and wait 3 minutes for the sample to cool. Pour the contents into the test cube.

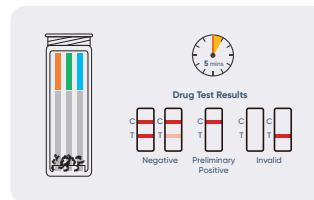

Read the results at 5 minutes.

Contact us to discuss the panel that meets your needs.

Rel.: 2025/04/24