



WELLlife™

# COVID-19/Influenza A&B Test

The WELLlife COVID-19/Influenza A&B Test Solution differentiates between SARS-COV-2, influenza A and influenza B antigens with a single test.

## Features and Benefits

- **Quality & Accuracy**

510(k) cleared for OTC (home) and professional use.

- **Ease of Use**

Differentiate between SARS-COV-2, influenza A, and influenza B antigens with an easy nasal swab.

- **Rapid & Convenient**

Results are available in just 10 minutes, and, kits that can be stored in 36°F-86°F (2°C-30°C).

- **Up-to-Date Variant Detection**

Detects multiple COVID-19 variants, including Omicron and the new XEC variant, as well as avian influenza (bird flu).

- **Extended Detection Window**

Offers a broader detection window for differentiation of SARS-COV-2, influenza A and influenza B antigens within four (4) days of symptom onset.

- **Pre-Loaded 12-Pack Retail Holder Design**

Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.

For Over-the-Counter Use



WELLlife™ COVID-19/Influenza A&B Home Test


For Professional Use



WELLlife™ COVID-19/Influenza A&B Test

# Application

For OTC Use

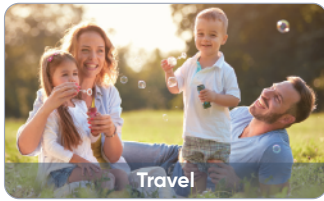


For Professional Use




## Quick 4-Step Test Process


- 1 Insert the swab 1/2 to 3/4 of an inch into nostril. Rotate the swab at least 5 times against the nostril wall. Repeat in other nostril.

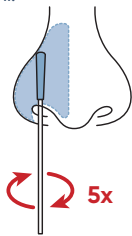


- 2 Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 10 circles.

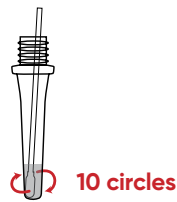

- 3 Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.

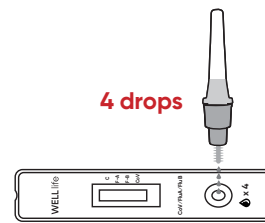


- 4 Start timer. Read results at 10 minutes.

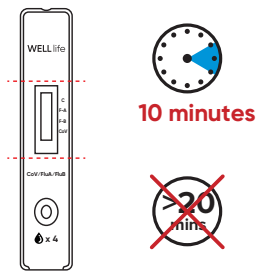


## Clear Interpretation

### Negative

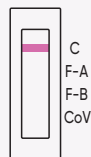


### Positive

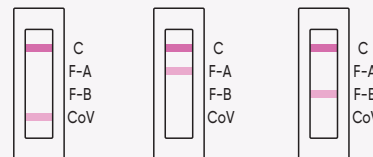

COVID-19 Positive    Influenza A Positive    Influenza B Positive

If the control line at "C" is visible and any other line or multiple lines at "F-A", "F-B" and/or "CoV" appear, the test is positive for that virus.

### Invalid

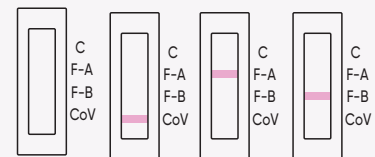

If a line is not visible at "C", even if any other line is visible in the results window, the result is considered invalid.

\* Please refer to QRI for detailed process interpretation.

## Ordering Information

Item Code	Description	Size	Classification
WCOVFLU-OTC-1	WELLife™ COVID-19/Influenza A&B Home Test 1T/Kit	1T/Kit	510(k) OTC
WCOVFLU-OTC-2	WELLife™ COVID-19/Influenza A&B Home Test 2T/Kit	2T/Kit	510(k) OTC
WCOVFLU-OTC-5	WELLife™ COVID-19/Influenza A&B Home Test 5T/Kit	5T/Kit	510(k) OTC
WCOVFLU-OTC-10	WELLife™ COVID-19/Influenza A&B Home Test 10T/Kit	10T/Kit	510(k) OTC
WCOVFLU-OTC-25	WELLife™ COVID-19/Influenza A&B Home Test 25T/Kit	25T/Kit	510(k) OTC
WCOVFLU-POC-5	WELLife™ COVID-19 / Influenza A&B Antigen Test	5T/Kit	510(k)
WCOVFLU-POC-10	WELLife™ COVID-19 / Influenza A&B Antigen Test	10T/Kit	510(k)
WCOVFLU-POC-25	WELLife™ COVID-19 / Influenza A&B Antigen Test	25T/Kit	510(k)

Rel.: 2025/01/21