



WELL life™

COVID-19 / Influenza A&B Home Test

FIRST Health Canada Class IV **approved**
COVID-19 / Flu A&B rapid Home (OTC) Test.

The WELL life™ COVID-19 / Influenza A&B Home Test is intended for the qualitative detection and differentiation of influenza A, influenza B and COVID-19 (SARS-CoV-2) viruses with a single nasal swab.

Features and Benefits

- **Reliable and Accurate**

Health Canada Class IV approved.

- **Ease of use**

Differentiate between SARS-COV-2, influenza A, and influenza B antigens with an easy nasal swab.

- **Rapid Results**

Accurate and reliable results are available in just 10 minutes.

- **Up-to-Date Variant Detection**

Detects multiple COVID-19 variants, including Omicron and the new JN.1 variant, as well as avian influenza (bird flu).

- **Extended Detection Window**


Offers a broader detection window for differentiation of SARS-COV-2, influenza A, and influenza B antigens within four days of symptom onset.

- **Suitable For Ages 2 and Above**

The test can be used independently by individuals aged 14 years or older. Adult collection is required for testing children 2-14 years old.

- **Pre-Loaded 12-Pack Retail Holder Design**

Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.

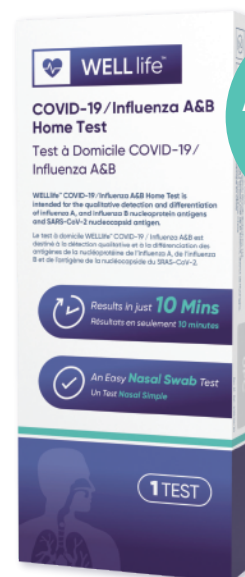

**HC
Approved**
for OTC
(Home) use



Application

For OTC Use



Travel



Back to school day




Back to work




Protecting the Vulnerable

Quick 4-Step Test Process


- 1** Insert the swab ½ to ¾ of an inch into nostril. Rotate the swab at least 5 times against the nostril wall. Repeat in other nostril.

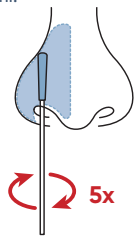


- 2** Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 10 circles.

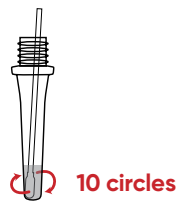

- 3** Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.

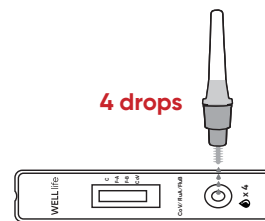


- 4** Start timer. Read results at 10 minutes.

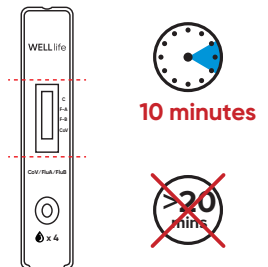


Clear Interpretation

Negative

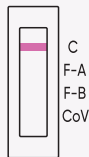

Positive

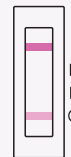

COVID-19
Positive






Influenza A
Positive



Influenza B
Positive

If the control line at "C" is visible and any other line or multiple lines at "F-A", "F-B" and/or "CoV" appear, the test is positive for that virus.

Invalid

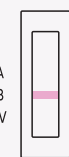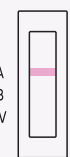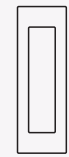

If a line is not visible at "C", even if any other line is visible in the results window, the result is considered invalid.

* Please refer to QRI for detailed results interpretation.

Ordering Information

Item Code	Description	Kit Size	Classification
CWV01P0001-1	WELL life™ COVID-19/Influenza A&B Home Test	1 Test	Class IV OTC
CWV01P0001-2	WELL life™ COVID-19/Influenza A&B Home Test	2 Tests	Class IV OTC
CWV01P0001-5	WELL life™ COVID-19/Influenza A&B Home Test	5 Tests	Class IV OTC
CWV01P0001-10	WELL life™ COVID-19/Influenza A&B Home Test	10 Tests	Class IV OTC
CWV01P0001-25	WELL life™ COVID-19/Influenza A&B Home Test	25 Tests	Class IV OTC

Rel.: 2025/05/01