



WELLlife™

COVID-19 Antigen Home Test

Health Canada Class IV approved for Over The Counter (Home) Use

The WELLlife™ COVID-19 Antigen Home Test provides a rapid and reliable way to detect the COVID-19 virus using nasal swabs from individuals showing symptoms. It's ideal for quick checks!

Why the WELLlife™ COVID-19 Antigen Home Test is the superior choice:

- **Reliable and accurate**
Health Canada Class IV approved.
- **Rapid Results**
Get results in just 10 minutes.
- **Easy to Use**
Identify COVID-19 with a simple nasal swab test.
- **Up-to-Date Variant Detection**
Detects multiple COVID-19 variants, including Omicron, keeping you informed on evolving virus strains.
- **Pre-Loaded 12-Pack Retail Holder Design**
Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage.
- **Suitable for Ages 2 and Above**
The test can be used independently by individuals aged 14 years or older. Adult collection is required for testing children 2-14 years old.
- **Extended Detection Window**
Offers a broader detection window for detection of the COVID-19 antigen within five days of symptom onset.



About WELLlife™

WELLlife™ is a leading brand of rapid testing products from Wondfo USA CO., Ltd. We're all about making health tests easy, fast, and hassle-free with our user-friendly test kits, putting reliable testing right at your fingertips. Backed by over 30 years of medical diagnostics expertise, a solid quality management system, and innovative R&D, WELLlife™ offers easy-to-use testing solutions you can count on. Trust WELLlife™ for your health needs!



Rel.: 2025/06/03



+1 (630) 468-2199 || info@wondfousa.com || WondfoUSA.com


Wondfo® || bringing science to life

Application

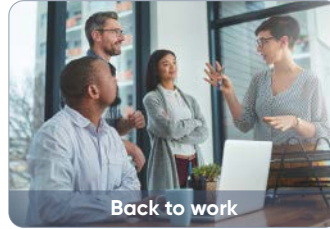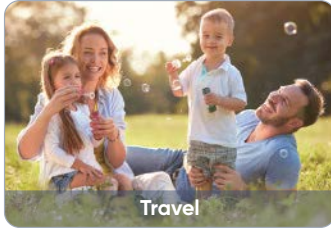

Quick 4-Step Test Protocol

- 1 GENTLY INSERT** the swab no more than 3/4 inch (1.5 cm) into the nostril. Slowly rotate the swab at least 5 times against the nostril wall.

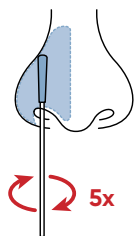


WARNING:
Be sure to rub BOTH nostrils with the same SWAB.


- 2 IMMERSE** the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 15 circles.



- 3** Invert the buffer tube and squeeze **4 drops** of test sample into the sample well on the test cassette.

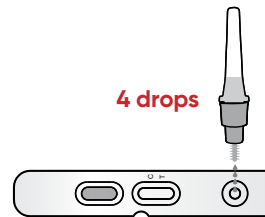

- 4** Start timer. Read results at **10 minutes**.

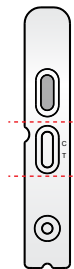


10 minutes



>20 mins

Clear Interpretation

Positive

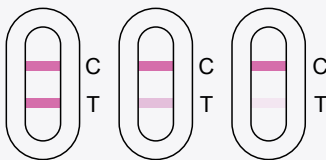


If the Control (C) line and the Test (T) line are visible, the test result is Positive. Any faint visible pink or purple test (T) line with the control (C) line should be read as positive.

Negative



If the Control (C) line is visible, but the Test (T) line is not visible, the test is negative.

Invalid

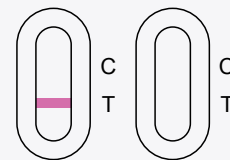

If a Control (C) line is NOT visible, DO NOT CONTINUE reading the results. It means that your test result is INVALID.

* Please refer to QRI for detailed results interpretation.

Ordering Information

Item Code	Description	Kit Size	Classification
CWL03P0001-1	WELLlife™ COVID-19 Antigen Home Test 1T/Kit	1 Test	OTC
CWL03P0001-2	WELLlife™ COVID-19 Antigen Home Test 2T/Kit	2 Tests	OTC
CWL03P0001-5	WELLlife™ COVID-19 Antigen Home Test 5T/Kit	5 Tests	OTC
CWL03P0001-10	WELLlife™ COVID-19 Antigen Home Test 10T/Kit	10 Tests	OTC
CWL03P0001-25	WELLlife™ COVID-19 Antigen Home Test 25T/Kit	25 Tests	OTC

