



SAFE life™

T-Dip Marijuana (THC) Multi-Level Urine Drug Test Panel

**5 THC
Cutoff Levels
15/50/100/200
/300 ng/mL**

Over 99% Accurate

More than 99% accurate in detecting THC with cut-off level as low as 15 ng/mL. It can detect THC usage in any form.

Semi-quantitative

Detect THC levels at 15 ng/mL, 50 ng/mL, 100 ng/mL, 200 ng/mL, and 300 ng/mL. With 5 different cut-off levels, you can accurately gauge where you stand and what level THC is appearing in your urine.




Easy to Use

Each test is sealed in an individual pouch for easy carry and to prevent contamination. It is easy to use and administer.

Results in 5 Minutes

Dip the test in urine for 10 seconds until dye rises up. Read the result at 5 minutes. 2 lines appear if negative; 1 control line only shows THC is present at that concentration. *NOTE if only 1 line shows in the T section, the result is invalid.

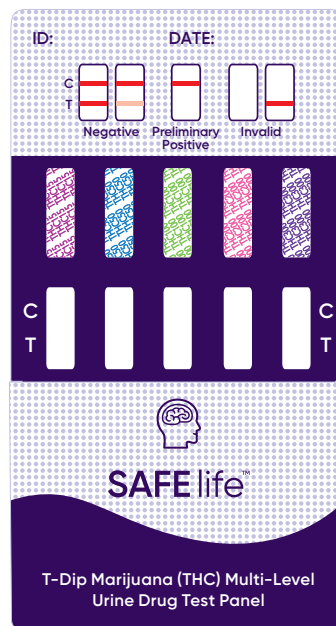

※ Kit Size: 25 / Case Size: 20

Item Code	Description	Complexity
WDTH-151	5-Panel Card; 15/50/100/200/300 THC Marijuana	Forensic Use Only

Rel: 2023/04/11