



WELLlife™

# Flu A&B Test

510(k) Cleared Flu A&B Rapid OTC (Home) and Professional use Test

The WELLlife™ Flu A&B Test detects both influenza A and B antigens with a nasal swab taken from individuals showing symptoms.

## Features and Benefits

- **Wide Range of Applications**

for OTC (Home) use and Professional use.

- **Rapid Results**

Get results in just 10 minutes.

- **Reliable and Accurate**

Differentiate between Flu A&B viruses with a simple nasal swab.  
**510(k) cleared for confidence.**

- **Wide Storage Temperature**

Detect both influenza A and B antigens with a simple nasal swab and obtain results in just 10 minutes. Kits can be stored at 36°F–86°F (2°C–30°C), allowing for easier storage and distribution.


- **Pre-Loaded 12-Pack Retail Holder Design**

Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.

- **Suitable for Ages 2 and Above**

The test can be used independently by individuals aged 14 years or older. Adult collection is required for testing children 2–13 years.

For Over-the-Counter Use

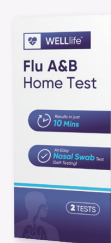

Retail holder

WELLlife™ Flu A&B Home Test

For Professional Use



WELLlife™ Influenza A&B Test

Rel.: 2025/08/28




+1 (630) 468-2199 || [wondfo@wondfousa.com](mailto:wondfo@wondfousa.com) || [WondfoUSA.com](https://WondfoUSA.com)

Wondfo®

bringing science to life

# Application

For OTC (Home) Use

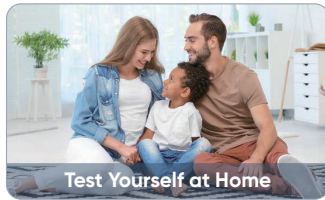

Test Yourself at Home



Safe for Testing Children



Back to Work Testing



Protecting the Vulnerable

For Professional Use



Laboratories



Primary Care



Urgent Care



Hospitals

## Quick 4-Step Test Protocol

- Insert the swab no more than 3/4 inch (1.5 cm) into the nostril. Rotate the swab at least 5 times against the nostril wall. Repeat in other nostril.
- Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 15 circles.
- Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.
- Start timer. Read results at 10 minutes. Do not read after 20 minutes.


## Clear Interpretation

### Negative




If a control 'C' line is visible, but the line at 'A' and 'B' is not visible, the test is **NEGATIVE**.

### Positive

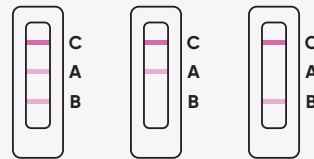

If the control line at 'C' is visible and any other line or multiple lines on 'A' and/or 'B' are visible, the test is **POSITIVE** for that virus.

### Invalid

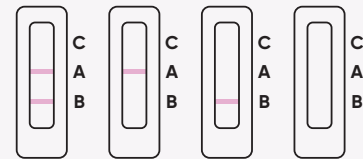

If a Control 'C' line is **NOT** visible, **DO NOT CONTINUE** reading the results. It means that your test results are **INVALID**.

\* Please refer to QRI for detailed process interpretation.

## Ordering Information

Item Code	Description	Size	Classification
WFLUAB-OTC-1	WELLlife™ Flu A&B Home Test	1 Test	510(k) OTC
WFLUAB-OTC-2	WELLlife™ Flu A&B Home Test	2 Tests	510(k) OTC
WFLUAB-OTC-5	WELLlife™ Flu A&B Home Test	5 Tests	510(k) OTC
WFLUAB-OTC-10	WELLlife™ Flu A&B Home Test	10 Tests	510(k) OTC
WFLUAB-OTC-25	WELLlife™ Flu A&B Home Test	25 Tests	510(k) OTC
WFLUAB-POC-25	WELLlife™ Influenza A&B Test	25 Tests	510(k) professional use
WFLUAB-CON-5	WELLlife™ Influenza A&B Test Control Kit	5 Tests	/

