



WELLlife™

COVID-19 Antigen Test

510(k) Cleared for OTC (Home) Use and Prescription/Professional Use

510(k)
cleared

The WELLlife™ COVID-19 Antigen Test provides a fast and reliable way to detect the COVID-19 virus using a simple nasal swab.

Features and Benefits

- **Wide Range of Applications**

510(k) cleared for OTC (Home) use and prescription use.

- **Up-to-Date Variant Detection**

Detects multiple COVID-19 variants, including Omicron, keeping you informed as virus strains evolve.

- **Easy to use and rapid results**

A single nasal swab offers accurate results in just 10 minutes, eliminating healthcare visits and reducing virus spread, ideal for the elderly and parents with young children.

- **Pre-Loaded 12-Pack Retail Holder Design**

Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage.

For Over-the-Counter Use



WELLlife™ COVID-19 Antigen Home Test

For Prescription Use



WELLlife™ COVID-19 Antigen Test Rx

Rel.: 2025/10/21




+1 (630) 468-2199 || wondfo@wondfousa.com || WondfoUSA.com

Wondfo®

bringing science to life

Application

For OTC (Home) Use

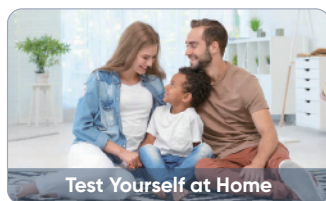

Test Yourself at Home



Safe Testing for Children




Back to Work Testing



Protecting the Vulnerable


For Prescription Use

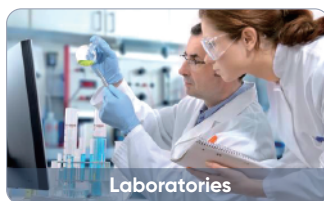

Laboratories



Primary Care

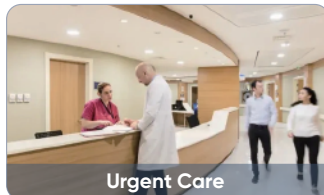


Urgent Care



Hospitals


Quick 4-Step Test Protocol

- 1 GENTLY INSERT** the swab no more than 3/4 inch into the nostril. Slowly rotate the swab at least 5 times against the nostril wall.

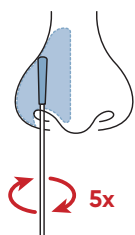


WARNING:
Be sure to rub BOTH nostrils with the same SWAB.

- 2 IMMERSE** the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 15 circles.

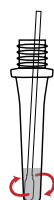

15 circles

- 3 Invert** the buffer tube and squeeze **4 drops** of test sample into the sample well on the test cassette.

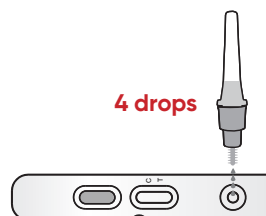

- 4 Start timer.** Read results at **10 minutes**.




10 minutes




Clear Interpretation

Positive

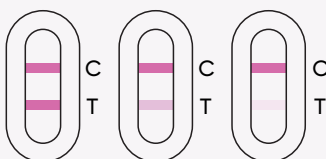


If the Control (C) line and the Test (T) line are visible, the test result is Positive. Any faint visible pink or purple test (T) line with the control (C) line should be read as positive.

Negative

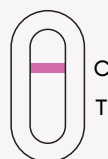

If the Control (C) line is visible, but the Test (T) line is not visible, the test is negative.

Invalid

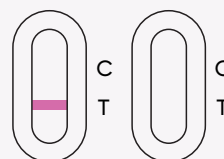

If a Control (C) line is NOT visible, DO NOT CONTINUE reading the results. It means that your test result is INVALID.

* Please refer to QRI for detailed results interpretation.

Ordering Information

Item Code	Description	Size	Classification
WL03P0001-1	WELLife™ COVID-19 Antigen Home Test	1 Test / Kit	510(k) OTC
WL03P0001-2	WELLife™ COVID-19 Antigen Home Test	2 Tests / Kit	510(k) OTC
WL03P0001-5	WELLife™ COVID-19 Antigen Home Test	5 Tests / Kit	510(k) OTC
WL03P0001-10	WELLife™ COVID-19 Antigen Home Test	10 Tests / Kit	510(k) OTC
WL03P0001-25	WELLife™ COVID-19 Antigen Home Test	25 Tests / Kit	510(k) OTC
WCOV-POC-25	WELLife™ COVID-19 Antigen Test Rx	25 Tests / Kit	510(k) prescription use
WCOV-CON-5	WELLife™ COVID-19 Antigen Test Rx Control Kit	5 Pos / 5 Neg	/

Rel.: 2025/10/21



+1 (630) 468-2199 || wondfo@wondfousa.com || [WondfoUSA.com](https://www.WondfoUSA.com)

Wondfo || bringing science to life