



WELLlife™

Flu A&B Home Test

Approved for Over-the-Counter use, no prescription required

The WELLlife™ Flu A&B Test detects both influenza A and B antigens with a nasal swab taken from individuals showing symptoms.

Features and Benefits

- **Wide Range of Applications**

Detects both influenza A and B antigens with a simple nasal swab. For OTC (Home) use. FSA & HSA eligible.

- **Rapid Results**

Get results in just 10 minutes.

- **Reliable and Accurate**

Differentiate between Flu A&B viruses, as well as the new Flu Subclade K Variant with a simple nasal swab. 510(k) cleared for confidence.

- **Wide Storage Temperature**

Kits can be stored at 36°F–86°F (2°C–30°C), allowing for easier storage and distribution.


- **Pre-Loaded 12-Pack Retail Holder Design**

Retail holders come pre-loaded with a total of 12 OTC 2ct packs for a total of 24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.

- **Suitable for Ages 2 and Above**

The test can be used independently by individuals aged 14 years or older. Adult collection is required for testing children 2-13 years.

For Over-the-Counter Use

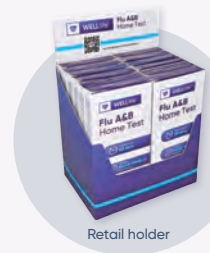

Retail holder

WELLlife™ Flu A&B Home Test


Application

For OTC (Home) Use




Quick 4-Step Test Protocol


1 Insert the swab no more than 3/4 inch (1.5 cm) into the nostril. Rotate the swab at least 5 times against the nostril wall. Repeat in other nostril.

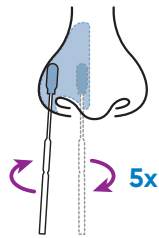


2 Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 15 circles.

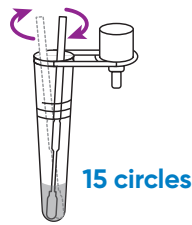

3 Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.

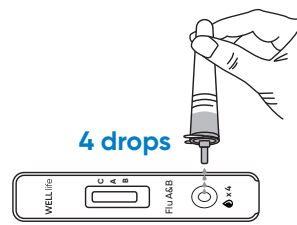

4 Start timer. Read results at 10 minutes. Do not read after 20 minutes.

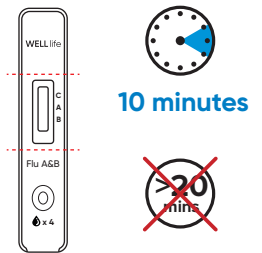


Clear Interpretation

Negative




If a control 'C' line is visible, but the line at 'A' and 'B' is not visible, the test is **NEGATIVE**.

Positive

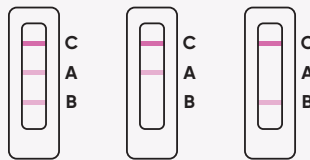

If the control line at 'C' is visible and any other line or multiple lines on 'A' and/or 'B' are visible, the test is **POSITIVE** for that virus.

Invalid

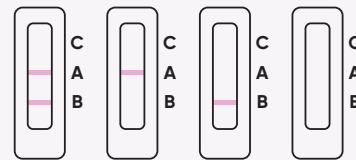

If a Control 'C' line is **NOT** visible, **DO NOT CONTINUE** reading the results. It means that your test results are **INVALID**.

* Please refer to QRI for detailed process interpretation.

Ordering Information

Item Code	Description	Size	Classification
WFLUAB-OTC-2	WELLlife™ Flu A&B Home Test	2 Tests / Box with 12 boxes per kit	Over-the-Counter (OTC) / Home use